

# SANTES® Gel

**Dermatological gel for external use containing sodium hyaluronate 0.33%**  
30 g tube.

## COMPOSITION

Sodium hyaluronate 99 mg, retinyl palmitate 15 mg, tocopheryl acetate 15 mg, 2-methyl-1,3-propanediol, glycerol, ammonium acryloyldimethyltaurate/VP copolymer, polysorbate 20, hydroxyethylcellulose, sodium benzoate, potassium sorbate, disodium edetate, purified water up to 100 ml.

## DEVICE CATEGORY

Multidose medical device.

SANTES® GEL is a water-based gel containing low and ultra-low molecular weight hyaluronic acid. Hyaluronic acid is a mucopolysaccharide abundantly present in the connective tissue, where it provides structural support and creates optimal conditions for the migration and proliferation of the cells involved in tissue repair. Numerous clinical studies have shown the ability of hyaluronic acid to accelerate and support re-epithelialisation and skin healing. Thanks to its hygroscopic properties, SANTES® GEL forms a protective barrier against external aggressors, restoring tissue tone and elasticity. The low and ultra-low molecular weight hyaluronic acid contained in SANTES® GEL ensures high bioavailability and efficacy, accelerating the physiological processes of wound healing and skin repair, while also moisturising the external genital area. The combination of retinyl palmitate (vitamin A) and tocopheryl acetate (vitamin E) provides antioxidant activity against free radicals, helping to protect the skin.

## CLINICAL INFORMATION

### INDICATIONS

Treatment indicated for adults as an adjuvant in tissue repair and regeneration, promoting the physiological healing of superficial and deep wounds (abrasions, grazes, burns, minor burns, surgical wounds, pressure sores and ulcers). Soothing and lubricating for the skin, it is indicated for the treatment of localised skin irritation and redness of any origin or nature, including those caused by physical treatments (radiotherapy, laser and skin peels). Also provides a moisturising effect for the external genital area in case of vaginal dryness.

### CONTRAINDICATIONS

Do not use in case of hypersensitivity to any of the ingredients.

### SIDE EFFECTS

No side effects have been reported to date. Inform your doctor or pharmacist of any undesirable effects suspected to be related to the use of the product.

### INCOMPATIBILITIES/INTERACTIONS

No interactions with other devices, substances or medicinal products are currently known.

### EFFECTS ON DRIVING OR OPERATING MACHINERY

No effects on the ability to drive vehicles or operate machinery have been reported.

## DOSAGE AND METHOD OF ADMINISTRATION

Wash hands thoroughly before and after applying the product. Unless otherwise advised by a doctor, apply the gel to the affected area 1-2 times daily, gently massaging until symptoms resolve, for a maximum of 30 days. Treatment may be repeated as needed.



## WARNINGS

The use of any topical product, especially for prolonged periods, may cause sensitisation; if this occurs, discontinue treatment and consult a doctor for the appropriate therapy. Under normal conditions of use, no cases of overdose have been reported. Follow proper cleaning and disinfection procedures for burns and wounds, including the surrounding tissue. Do not use on severe or infected burns; in such cases consult a doctor. Close the tube tightly after each use. This product is intended for external use only. Do not use the product after the expiry date shown on the packaging. Do not use the product if the packaging is not intact. Keep out of the reach of children. Do not swallow. Any serious incident that occurs in relation to this medical device must be reported to the manufacturer and to the competent authority of the Member State in which the user resides. Use within 6 months from opening.

## INTERACTIONS

No interactions are known between SANTES® GEL and any concomitant therapies.

## SPECIAL PRECAUTIONS FOR STORAGE


Store at room temperature ( $\leq 25$  °C), away from light and heat sources.

## PACKAGING

30 g tube.

## DISPOSAL

There are no special conditions for disposal. Refer to local regulations.

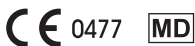

Paper leaflet PAP 22

Recycling - Please check the regulations in your region



Lo.Li. Pharma S.r.l.  
Via Sabatino Gianni, 14- 00156 Rome - Italy

